FILL Magic SHAPE

Resina Modeladora Fotopolimerizável
Resina Modeladora Fotopolimerizable
Light-Curing Shaping Resin

PORTUGUÊS

1.Descrição:

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ENGLISH

1.Description

Fill Magic Shape is a light curing, radiopaque, low-viscosity resin, with 30%

FILL Magic SHAPE

load, which facilitates the sculpting and contouring of composite resin restorations when they become dry and/or sticky during incremental layers insertion.

2. Indication

Fill Magic Shape is indicated to the humidification of instruments (brush and/or spatula) during composite resin sculpting stage, or when it is desired to smooth the surface in the last layer of resin before light curing, facilitating the sculpting and contouring of composite resin restorations.

3. Composition

Bisphenol A-glycidyl methacrylate; Triethylene glycol dimethacrylate (TEGDMA); Ethyl 4-(dimethylamino)benzoate; Photoinitiator; BHT (butylhydroxytoluene), Silicon Dioxide; Glass Powder

4. Presentation

02 syringes containing 1.2g each
02 cannulas for application
01 Instructions for Use

5. Application techniques

Remove the syringe cap and attach the cannula to the syringe.

Apply a small and discreet amount on the tip of the spatula, brush, or on the surface of the not yet polymerized composite.

Make the sculpture or anatomical shape in the composite resin and photo-activate it according to the manufacturer's instructions.

6. Precautions, Contraindications, Restrictions and Warnings

6.1. Precautions

This product was manufactured to be used only as stated in the Instructions for Use. Use other than that described in the instructions for Use is the sole and exclusive responsibility of the professional who uses it.

This product is sensitive to light. Therefore, overly bright ambient lights can cause premature polymerization.

Always keep the product closed when not using.

Once out of the syringe, the product has to be used quickly.

When not in use replace the cap tightly.

6.2. Restrictions and Warnings

FOR PROFESSIONAL USE ONLY

Keep out of the reach of children.

All dental material must be used only by authorized dentists and dental laboratories.

6.3. Contraindications

There are no contraindications when **Fill Magic Shape** is used by a dental professional, in accordance to the Instructions for Use. However, we recommend not using it in patients who have a history of allergy to any of its components, or even when the stipulated technique cannot be applied.

7. Storage Conditions

Fill Magic Shape serves the purpose for which it was designated if stored at room temperature (15°C - 30°C) in dry and cool place, protected from light and heat.

After using, keep the product in its original packaging and closed.

Direct light on the product can change its properties.

High ambient temperature requires cooling storage.

The product must be taken off the refrigerator in order to return to room temperature before using.

Fill Magic Shape must not be stored with products which contain phenol (phenol, clove oil, copal varnish) or zinc oxide cements with eugenol, as these prevent or affect polymerization.

8. Expiration Date

Fill Magic Shape has a 36-month shelf life from manufacturing date, even after opened, as long as it is kept in its original packaging and in adequate storage conditions.

The expiration date shown on the packaging represents the maximum period for using the product. Do not use the product after the expiration date.

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