## PORTUGUÊS

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## ENGLISH

**1. DEFINITION**  
 Magic Bond Universal is a photopolymerizable agent, responsible for the bond between the dental structure (enamel and dentin) and other restorative materials, aiming adhesive restoration technique. It can be applied with the self-conditioning technique, selective conditioning, or total conditioning.

**2. PRESENTATION**  
 01 bottle of Magic Bond Universal, which may contain 1mL; 2mL; 3mL; 4mL; 5mL; 6mL; 7mL; 8mL; 9mL; 10mL; 11mL or 12mL.  
 01 Printed or digital usage instruction which can be accessed through a QR code scan on the cartridge.

**3. COMPOSITION**  
 Composition: Monomers (Bis-GMA, GDMA, UDMA), BHT, Ethyl-4-Dimethyl Amino Benzoate, 2-Hydroxyethyl Methacrylate, Adhesiveness promoters (Methacrylate type), 2,4,6-trimethylbenzoyldiphenyl phosphine oxide, Ethyl Alcohol, Photoinitiators, and Fluorescence agent.

**4. INDICATIONS**  
 Magic Bond Universal is indicated for:  
 Direct restorations of all classes with composites; Adhesive bonding of indirect restorations (inlays, onlays, veneers, crowns, and intracanal posts) in fiberglass, conditionable or non-conditionable ceramics, composites, and metals;  
 Adhesive repairs on ceramics and composites; Dentin sealing.

**5. APPLICATION TECHNIQUES**  
 • Self-conditioning: Apply Magic Bond Universal directly to the enamel and dentin.  
 • Selective conditioning: Firstly, condition the enamel. Then, apply Magic Bond Universal to the enamel and dentin.  
 • Total conditioning: Firstly, condition both the enamel and dentin. Then, apply Magic Bond Universal to the enamel and dentin.

**6. HOW TO USE**  
 6.1. Direct Restoration on Enamel and/or Dentin  
 6.2. Dental Element Preparation  
 Conduct prophylaxis with pumice stone and water. If necessary, record tooth color and proceed with intraoral anesthesia.  
 6.2.1. Cavity Preparation  
 Prepare the oral cavity according to the principle of the adhesive restoration technique.  
 For isolation, it's recommended to use a rubber dam. Protect the pulp by applying a small amount of hard drying calcium hydroxide cement.  
 Finish the enamel with a diamond drill bit.  
 If the enamel is not cut, treat it with Magic Acid 37% phosphoric acid gel or clean with pumice stone. Then, rinse well and thoroughly with water and dry the enamel carefully.

6.2.2. Conditioning  
 a) Acid conditioning is not required in the self-conditioning technique.  
 b) For the selective conditioning technique: apply only on the enamel.  
 c) Total conditioning technique: apply on both the enamel and dentin. For b) and c), apply the Magic Acid 37% phosphoric acid gel according to the usage instructions. Then, rinse well and ensure that no residues are left. Dry the cavity with oil-free air. Important: Do not dry out the dentin; preferably, dry with absorbent paper.  
 6.2.3. Application  
 Shake the bottle well before using. Release a drop of Magic Bond Universal into the mixing container and apply to the cavity using Technobrush. Rub the surface for 20 seconds and carefully dry with oil-free air for 5 seconds. Polymerize for 10 seconds (photopolymerization with halogen or LED with a light intensity > 800mW/cm<sup>2</sup>). A lower light intensity requires a longer exposure time. Note: Avoid accumulating adhesive agents. This can impair the accuracy of the adjustment.

**7. ADHESIVE BONDING OF INLAYS, ONLAYS, VENEERS AND CROWNS**  
 7.1. To use Magic Bond Universal, it is essential to ensure that enough light is penetrated throughout the entire restoration, according to the manufacturer's instructions.  
 7.2. The procedure and preparation for restoration depends on the material and must follow the manufacturer's usage instructions.  
 7.3. Cavity preparation should be carried out according to step 5.1.1 of this instruction.  
 7.4. Conditioning should be done following step 5.1.2 of this instruction.  
 7.5. Application should be done as step 5.1.3 of this instruction.  
 7.6. Cementation must be done following the manufacturer's usage instructions.

**8. ADHESIVE BONDING TO OTHER MATERIALS**  
 8.1. Ceramic Preparation  
 According to the manufacturer's usage instruction.  
 8.2. Metal and Amalgam Preparation  
 Follow the manufacturer's usage instruction. Then, clean with a steam cleaning device or with acetone and dry with oil-free air.  
 8.3. Preparation of Non-conditionable Ceramics  
 Remove material from the surface with a diamond drill or a sandblast with 50µm aluminum oxide. Clean the surface (e.g., with phosphoric acid), rinse, and dry with oil-free air.  
 8.4. Application  
 Apply Magic Bond Universal to the surface using Technobrush.  
 Rub the surface for 20 seconds and carefully dry with oil-free air for 5 seconds. Polymerize for 10 seconds (photopolymerization with halogen or LED with a light intensity > 800mW/cm<sup>2</sup>). A lower light intensity requires a longer exposure time. Note: Avoid the accumulation of adhesive agents. This can impair the adjustment's accuracy.

**CONTRAINDICATIONS**  
 There are no contraindications when Magic Bond Universal is used by a dentistry professional following the usage instructions. However, for patients with a history of allergy to any of its components, we recommend not using it. Also, it should not be used when the advised technique cannot be applied.

**PRECAUTIONS**  
**FOR PROFESSIONAL USE**  
 • The components of Magic Bond Universal can cause sensitivity in predisposed individuals.  
 • Do not store it alongside with materials containing eugenol or clove oil, which can inhibit the polymerization of Magic Bond Universal.  
 • In case of direct contact with oral mucosa, rinse with plenty of water. If in contact with eyes, rinse thoroughly with water. Please seek medical assistance for assessment and any necessary follow-up treatment.  
 • After dispensing/using the product, immediately close the bottle and the mixing container.  
 • The use of MAGIC BOND UNIVERSAL is not recommended in cases of inadequate oral hygiene.  
 • The use of Magic Bond Universal is not recommended if it is impossible to keep dry the working area.

**WARNINGS**  
 • Keep away from children!  
 • Discard empty packaging in accordance with the guidelines for disposing of materials, products, and packaging used in dental offices or laboratories.  
 • For the best hygiene conditions when using Magic Bond Universal, it is recommended to use disposable applicators like Technobrush.

**SELF LIFE**  
 Under proper storage conditions, Magic Bond Universal has a shelf life of 36 months, even after being opened, as long as it is kept in its original packaging, properly closed and stored according to storage instructions.  
 The expiration date, as indicated on the packaging, represents the maximum period for the product usage. Do not use after the expiration date. Immediately close the bottle after use to prevent volatilization or polymerization of Magic Bond Universal.

**STORAGE**  
 Store Magic Bond Universal between 2°C and 30°C. If stored in the refrigerator, allow the product to reach room temperature (15°C - 30°C) before use. Frequent exposure of Magic Bond Universal to temperatures above 30°C may reduce its lifespan and/or alter its physical-chemical characteristics. Do not expose to direct sunlight or other heat sources. Do not store it alongside with materials containing eugenol or clove oil, which can inhibit the polymerization of Magic Bond Universal. Keep the bottle tightly closed immediately after use.

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