Barreira Gingival

Barreira gingival fotopolimerizável
Light-curing gingival barrier
Barrera gingival fotocurable

PORTUGUÊS

1. Descrição

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ENGLISH

1. Description

The hydrogen-peroxide-based whitening gel can irritate the tissues and cause great discomfort when in contact with the gum, thus compromising the whitening treatment. The gingival barrier use is recommended in order to avoid this discomfort, guaranteeing better performance in dental whitening treatments.

The Vigodent Gingival Barrier is an exclusive light-cured reflective resin barrier. Due to its light-reflecting properties, Vigodent's Gingival Barrier is ideal for gingival protection when whitening a tooth or an entire arch. The Gingival Barrier has a methacrylate base, and has been developed not only to act as an effective barrier, but also to be of easy removal from embrasures and hollows. Its unique syringe application system ensures control, ease of use, and minimal waste.

2. Use Indications

It is indicated as a substitute for dental rubber dam to protect the gingival tissue and when there is irritation risk due to contact with products used in clinical procedures, such as tooth whitening. It is also indicated as an adjunct in absolute isolation for better sealing in cavity preparations or endodontic treatments.

3. Composition

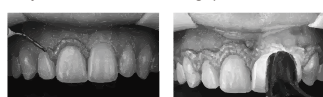
UDMA, HEMA, silica, pigmentos, camphorquinone, ethyl 4-dimethylaminobenzoate

4. Contents

- 1 syringe containing 2g of product and 3 application tips
- 1 instructions manual for the professional

5. Instructions for use

- 1- Dry the gum and teeth corresponding to the area to be protected.
- 2- Use the application tip to set a product layer of approximately 01mm of width on the gingiva, including the previously dried papillae (Fig. 1).
- 3- Light cure the resin using LED or halogen light-curing light for about 20-30 seconds. Proceed with polymerization immediately after application of the resin to prevent any possible irritation (Fig. 2).
- 4- Make sure the product covers the entire surface of the gingiva, and that it is completely cured. Avoid testing its firmness by forcing or trying to detach it, as this can cause the irritant substance to seep in when applied.
- 5- To remove the product simply detach it from the gum, pulling it out until it easily comes off in whole or in large pieces.

6. Precautions, Contraindications, Restrictions and Warnings

6.1. Precautions

- Avoid applying the product to the gum of patients who have recently undergone any type of oral surgical procedure, or those who have any kind of gingival irritation.
- Incomplete polymerization of the product or excessive time may lead to mild gingival irritation in some cases.

6.2. Warning

- The product has methacrylate monomers, which can cause allergic reactions or mild irritation to patients who are sensitive to these substances. Stop using the product in case of allergic reactions.

- Observe the expiration date. Never use the product if it has expired. Follow local legislation to dispose of the product. Keep it out of the reach of children.

6.3. Contraindications

There are no contraindications when the Gingival Barrier is used by a dental professional in accordance with the instructions for use. However, we recommend not using it in patients who have a history of allergy to any of its components, or when the adequate technique cannot be applied.

7. Expiration

Under proper storage conditions, Gingival Barrier has a 24-month shelf life, even after opened, since it's kept in your original packaging and properly closed. The expiration date displayed on the packaging represents the maximum period for using the product. Do not use it after the expiration date.

8. Storage and Transport Conditions

- Always keep the product in its closed packaging.
- Protect from direct sunlight.
- Store the product at temperatures from 5°C to 28°C.

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