Dessensibilizante

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ENGLISH

1. Product Description

Dessensitizer is a low viscosity neutral pH topical desensitizing gel with dual desensitizing action, made from Potassium Nitrate (neural action) and Sodium Fluoride (occlusion of the dentinal tubules) with boosted action over sensitivity. It is indicated for use before, during, and after tooth whitening and whenever patients report sensitivity.

1.1. Indications

Vigodent Desensitizer Gel forms a stable film on sensitive teeth, acting to seal the dentinal tubules of exposed dentin areas and other areas where post-operative sensitivity is a concern.

It is used only by professionals in office.

1.2. Presentation

Refill
01 syringe containing: 2g
01 application tip
01 Instructions for use

2. Instructions for use

Vigodent Desensitizer is a gel indicated to treat and prevent tooth hypersensitivity before, during and/or after the in-office tooth whitening procedure, caused by gingival recession or periodontal treatment.

Proceed with dental cleaning and prophylaxis steps to remove dental residue or plaque

If necessary, isolate the field to be treated

Dispense initial portion of Desensitizer for pressure removal.

Dry the entire tooth surface to be treated with Desensitizer.

Apply the Desensitizer with the applicator tip directly on the dental faces to be treated. If necessary, use Technobrush as an aid.

Let it act for 10 minutes

Remove the gel with a dry and sterile gauze

Wash thoroughly with jets of air and water

Proceed with subsequent procedures

4. Precaution, Warnings and Contraindications

4.1. Precaution

Keep it out of the reach of children.

This product is NOT intended for overnight use.

Do not smoke or eat while using this product.

This product is NOT intended for use in pregnant women.

If problems occur, interrupt the treatment and notify your dentist.

Always keep it away from heat/sunlight. It should be stored at room temperature.

Patients should not swallow large volumes of Desensitizer. Excess material and residue after removal should be brushed off and washed away. Do not swallow.

This product is intended for professional use only

Review product instructions, precautions and safety data sheets before initiating treatment.

Use only as directed.

Make sure the patient has no known allergies to the treatment materials.

4.2. Warnings

Avoid deliberate and excessive ingestion of the product. Ingestion of large amounts of the product may cause nausea.

In case of ingestion, drink plenty of water and seek medical attention in case you feel any discomfort.

4.3. Contraindications

There are no contraindications when Desensitizer is used by a dental professional in accordance with the instructions for use. However, we recommend not using it in patients who have a history of sensitivity and/or allergy to any of its components.

5. Expiration

Desensitizer has a 24-month shelf life, from manufacturing date, even after opened, as long as it is kept in its original and properly closed packaging, and stored under proper conditions. The expiration date displayed on the packaging represents the maximum period for using the product.

Do not use it after the expiration date.

6. Product Storage

Always keep this product in its closed packaging.

Protect from direct sunlight.

Store the product at temperatures from 5°C to 28°C.

There are no special handling conditions as long as the Desensitizer is used by a dental professional, according to the instructions for use.

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