PORTUGUÊS

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ENGLISH

1. Product Description

Fill Magic Tempo is a temporary lightcuring restorative material, based on dimethacrylate resin, used in procedures that require the protection of tooth prior to placement of the definitive prosthetic work. **Fill Magic Tempo** is applied incrementally or directly on the prepared tooth, rebuilding its shape and aesthetic function. Can be used in anterior and posterior teeth; it is sufficiently strong to withstand the wear and impact of masticatory forces and flexible enough to be easily removed before the placement of the definitive prosthetic element.

1.1. Indications

Temporary restorative in inlay/onlay procedures, crowns, bridges, re-ramping of prefabricated, temporary, provisional acrylic sealing in endodontics and implantology.

1.2. Composition

UDMA, TEGMA, EDAB, BHT, fotoiniciador, loads.

1.3. Presentation

Fill Magic Tempo replacement:

- 01 syringe containing 3g
- 01 instruction for use

2. Application Techniques

NOTE 1: After preparing the cavity and performing the definitive impression, the cavity must be kept wet. Depending on the extension of the prepared area, there might be a need for using a matrix band or

build up the restoration through successive increments.

1. Apply **Fill Magic Tempo** to the cavity with the aid of a spatula and sculpt the anatomy properly, leaving minimal excess of material.
 2. Ask the patient to close the jaws in occlusion and check for occlusal and incisal interference prior to lightcuring.
 3. Light cure for 40 seconds with curing light device with intensity at least 300mW/cm² (400-500nm). If necessary, light cure extensive areas of larger cavities during 40 seconds per side involved in the cavity preparation.
 4. Remove the matrix (when used) and check the final occlusion.
- NOTE 2: For complex cavities preparations, you can opt for the temporization with **Fill Magic Tempo** applied over Temp Cem NE or Liner.

5. Adjust contour with Diamond tips, if needed.

3. Precautions, Restrictions and Warnings

3.1. Precautions

- **Fill Magic Tempo** shall be used by dentists who will decide the duration of treatment depending on the case.
- There is no danger associated to the ingestion of **Fill Magic Tempo** due to the small amount applied and the absence of toxicity.
- In the case of a direct contact with the eyes, rinse thoroughly with water for about 10 minutes and consult an eye specialist.
- Keep out of reach of children.
- Avoid direct exposure of the material to sunlight or other bright light sources.
- This product is manufactured to be used only as indicated in the instructions for use. The use of form other than described in the instructions for use, are the sole and exclusive responsibility of the professional use.
- After using the material, it must be properly closed.
- All dental materials must be used solely by dentists and dental laboratories authorized.

3.2. Restrictions and Warnings

There is no contraindication, when **Fill Magic Tempo** is used by a dental professional according to the instructions for use. However, in patients with previous history of allergy to the components of this product, it is recommended not to use.

4. Validity

The expiration date on the packaging represents the maximum period for the use of the product. Do not use after the expiration date. Under appropriate conditions of storage, **Fill Magic Tempo** has a validity of 36 months.

5. Storage

- Must remain in their original container and closed after use.
- Store at room temperature (15°C - 30°C), in a dry place, protected from light and heat.
- Direct light on the product can change the properties of the **Fill Magic Tempo**.

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