CIMENTO ENDODÔNTICO

PORTUGUÊS

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ENGLISH

1.Product Description

ENDODONTIC CEMENT is a material for filling root canals, which works through the basic reaction between its main components: Zinc Oxide and Eugenol. Vigident ENDODONTIC CEMENT is characterized by good tissue tolerance, radiopacity, impermeability and fine granulatin, providing a homogeneous, creamy and lump-free mixture.

2.Indications

Indicated for filling root canals.

3.Presentation

Endodontic Cement - Powder

01 bottle containing 12g

01 measuring spoon

Endodontic Cement - Liquid

01 bottle containing 10mL

KIT

01 Bottle Endodontic Cement - Powder, containing 12g

01 Bottle Endodontic Cement - Liquid, containing 10mL

01 measuring spoon

4.Composition

Powder: Zinc oxide, dimerized resin, bismuth subcarbonate, barium sulphate, sodium borate and zinc acetate.

Liquid: Eugenol and Almond Oil.

5.How to use

How to use: After completing the instrumentation, clean and dry the root canal; Select the appropriate gutta-percha cone for the case; Dispense 3 drops of liquid for 3 spoonfuls of powder (approximately 0,6g of powder) into the previously sterilized mixing plate; Spatulate the product, gradually incorporating the powder into the liquid until a smooth, homogeneous and consistent mixture is obtained.

Then introduce the endodontic cement into the

root canal according to the conventional endodontic technique selected.

NOTES: ENDODONTIC CEMENT is soluble in ether, xylol and chloroform. The amount of 3 tablespoons of powder to 3 drops of liquid should be respected, in order to ensure optimal handling of the product without affecting its physical and chemical characteristics.

6.Precautions, Restrictions and Warnings

Precautions

- This product is intended to be used only as described in the instructions for use. Any use other than that described in the instructions for use is the sole responsibility of the professional using it.
- Avoid direct exposure of the material to sunlight or other sources of intense light.
- All dental materials should only be used by authorized dentists and dental laboratories.
- The liquid contains eugenol, which can cause irritation to the skin, oral mucosa and eyes.
- In the event of prolonged contact, rinse the affected area with plenty of water.

- In more serious cases of eye involvement, seek medical attention.

Restrictions and Warnings

- Keep out of reach of children;

- Contains Eugenol;

- FOR PROFESSIONAL USE;

- Single use product;

- No reprocessing;

- Disposal of the product must be carried out in accordance with the disposal of waste materials from dental practices and/or in accordance with local legislation, ensuring that it is completely disposed of in order to prevent its reuse and environmental damage.

7. Contraindications

There are no contraindications when ENDODONTIC CEMENT is used by a dentist in accordance with the instructions for use. However, patients with a history of allergy to the components of this product should not use it. Products containing essential oils, such as eugenol, can cause allergic reactions in susceptible people. If there is a history of allergy to essential oils, do not use.

8. Storage

ENDODONTIC CEMENT fulfills its intended purpose if stored at room temperature (15°C - 30°C), in a dry place, protected from light and heat.

The product must remain in its original, unopened packaging after use.

Avoid direct exposure of the material to sunlight or other sources of intense light.

Unsuitable storage conditions may compromise the product's performance.

9. Shelf life

Under appropriate storage conditions, ENDODONTIC CEMENT has a shelf life of 36 months even after opening, provided it is kept in its original packaging and properly sealed.
The expiry date on the packaging represents the maximum period of use for the product.
Do not use after the expiration date.

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