FILL Magic ORTHO

Adesivo Ortodôntico com Carga
Adhesivo Ortodontico con Carga
Orthodontic Adhesive with Load

PORTUGUÊS

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ENGLISH

1. Description of the product

Fill Magic Ortho is an orthodontic adhesive with load, suitable for the fixing brackets. **Fill Magic Ortho** doesn't require orthodontic mixture, is easy to apply and its composition promotes excellent adhesion of the support to the tooth.

1.1. Indication

Fill Magic Ortho is indicated for fixing metal orthodontic brackets, especially for the technique of wire and plastic brackets fixation.

1.2. Composition

Mixture of Monomers, Charges, Titanium Dioxide, Fluorine and Pigments.

1.3. Presentation

Fill Magic Ortho: 01 syringe (weight NET vide-tag) and 01 instruction for use.

2. Application Techniques

1. Clean the surface of the enamel with prophylactic paste free from fluoride, oil or flavoring. Clean with water and dry with an air jet.
2. Make conditioner dental Magic Acid 37% on dental surfaces, after clinical assessment, up to 60 seconds. Clean with water and dry with an air jet.
4. Apply a thin layer of **Fill Magic Ortho** in the base bracket and

place it on the prepared tooth.

5. Make a first removal of excess before the polymerization.
6. On the proper placement of the bracket on the tooth surface, polymerize in cervical and incisal for 20 seconds each, with device type LED or halogen curing light.
7. Remove the excesses around the bracket, with the help of a fine finish tip.
8. Level wire can be done after polymerization.

3.1. Precautions, restrictions and warnings

3.1. Precautions

- Avoid direct exposure of the material from the light of the sun or other bright light sources.
- **Fill Magic Ortho** should not be used or stored with products that have phenol in their composition (fenol, clove oil, varnishcopal) or zinc oxide and eugenol cements because these materials prevent or affect the polymerization.
- Discoloration may occur around the brackets using cationic mouthwashes as well as developers of plate and chlorhexidine, over an extended period.
- After using the material, it must be properly closed.
- Keep out of the reach of children.
- This product is designed for use only as set forth in the instruction for use. The use in such a way that described in the operating instructions is the sole and exclusive responsibility for the professional use.
- All dental materials must only be used by dentists and Authorized dental laboratories.

3.2. Restrictions and Warnings

There is no contraindication if **Fill Magic Ortho** is used by a professional dental orthodontic according to instructions. However, in patients with previous history of allergy to the components of this product, do not use.

4. Validity

- 36 months after the date of manufacture. Do not use after expiration date.
- The expiration date on the packaging represents the maximum period for using of the product.
- Inadequate storage conditions can compromise the product performance.

5. Storage

- Must remain in closed original container after use.
- Store at room temperature (15°C - 30°C) in a dry place, protected from light and heat.
- Direct light can change the properties of **Fill Magic Ortho**.

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