FILL Magic DUAL Cement

PORTUGUÊS

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ENGLISH

1. Description:

Fill Magic Dual Cement is a (dual curing) resin cement, multifunctional and self-polymerizable,

radiopaque, indicated for inlay and onlay restorations, porcelain or ceramic, metallic crowns, fixed implants, adhesive prosthodontics and endodontic core.

1.1. Composition:

Base: Bis-GMA, Bis-EMA, TEGDMA, BHT, EDAB, camphoroquinone, borocaluminosilicate and pigments.

Catalyst: TEGDMA, Bis-EMA, Bis-GMA, BHT, benzoyl peroxide and borocaluminosilicate.

1.2. Indications:

- Inlay or onlay cementation of porcelain, ceramics or composite veneers.
- Cementation of full-coverage crowns, fixed implants, adhesive prosthodontics and endodontic core.

1.3. Presentation:

1x syringe of Fill Magic Dual Cement Base with 2,5 g;

1x syringe of Fill Magic Dual Cement catalyst with 2,5 g;

1x spatulation block;

1. Instructions for Use.

2. Instructions for Use:

2.1. Application techniques:

If the laboratory did not apply any conditioning treatment, proceed as follows:

Metal pieces: Microconditioning of surfaces should be performed by sandblasting with 50 µm aluminum oxide or electrolytic conditioning.

Porcelain pieces: Apply a 10% hydrofluoride acid to the porcelain veneer's inner surfaces. Then apply the adhesive with a brush, let it act for 1 minute and dry thoroughly with air.

2.2. Inlay and onlay cementation:

Remove the temporary material, completely remove the cement from the cavity preparation and perform prophylaxis. Perform absolute insulation. Perform the enamel acid conditioning technique, according to clinical evaluation and the manufacturer's technical recommendation.

Note 1: Whenever only enamel surfaces are involved, 30-second acid conditioning is recommended, followed by water rinse and air stream drying. Apply the Magic Bond adhesive, remove excess and light-cure for 20 seconds to obtain a thin layer of adhesive.

When both enamel and dentin are involved, a full acid conditioning technique (enamel, 30 seconds; dentin, 15 seconds) is recommended. Then, rinse

with water and dry the surface gently, leaving the enamel dry and the dentin wet.

Apply Magic Bond DE fluoride adhesive for 20 seconds and evaporate the solvent using a 5-second stream of air, proceeding with photoactivation for 20 seconds. Dispense equal parts of Fill Magic Dual Cement base and Fill Magic Dual Cement catalyst onto the spatulation block. Mix the base and the catalyst until a uniform color is obtained, after approximately 10 seconds.

Dispense a fraction of the mixture of Fill Magic Dual Cement base and Fill Magic Dual Cement catalyst into the piece, and the other part on the cavity preparation, place it in position by applying slight pressure to provide better adaptation.

Light-cure for 3 seconds and then remove the excess occlusive proximal and cervical margins using the appropriate tool.

After removing all excess, light-cure the Fill Magic Dual Cement on each of the dental faces for 40 to 60 seconds. Complete with the finishing step, checking the occlusion and promoting polishing using the ONESTEP Silicone Polishing Tips from Coltene.

2.3. Cementation of adhesive prosthodontics

Remove the provisional restoration, completely eliminate the fixation cement in the cavity and perform prophylaxis. Perform absolute insulation. Promote the technique of acid conditioning on the enamel, according to clinical evaluation and the manufacturer's technical recommendation. Dispense onto the spatulation block equal parts of Fill Magic Dual Cement base and Fill Magic Dual Cement catalyst. Mix the base and the catalyst to a uniform color, approximately 10 seconds.

Place a fraction of the mixture of Fill Magic Dual Cement Base and Fill Magic Dual Cement Catalyst into the piece, and the other part in the preparation. Fit the piece into the preparation by applying gentle pressure in order to promote a better adaptation. Light-cure for 3 seconds and then remove the excess occlusive proximal and cervical margins using the appropriate tool.

After removing all excess, light-cure the Fill Magic Dual Cement on each of the dental faces for 40 to 60 seconds. Complete with the finishing step, checking the occlusion and promoting polishing using the ONESTEP Silicone Polishing Tips from Coltene.

2.4. Crown cementation:

Remove the provisional restoration, fully eliminate the fixation cement from the cavity preparation and perform prophylaxis. Perform absolute insulation. Promote the technique of acid conditioning on the enamel, according to clinical evaluation and the manufacturer's technical recommendation.

Dispense equal parts of Fill Magic Dual Cement base and Fill Magic Dual Cement catalyst onto the spatulation block. Mix the base and the catalyst to a uniform color, i.e. approximately 10 seconds.

Place part of the mixture on the inner crown walls and, if necessary, on the remaining dental core or structure and position it correctly.

Light-cure for 3 seconds and then remove the excess occlusive proximal and cervical margins using the appropriate tool.

After the excesses have been removed, light cure the Fill Magic Dual Cement with a light curing agent for about 40 to 60 seconds on each face.

Note 2: In the case of metallic crowns the curing will occur only by the chemical activation of the product due to the blockage of passing light. Complete with the finishing step, checking the occlusion and promoting polishing using the ONESTEP Silicone Polishing Tips from Coltene.

2.5. Cementation of facets:

Perform absolute insulation. Promote the technique of acid conditioning on the enamel, according to clinical evaluation and the manufacturer's technical recommendation.

Dispense equal parts of Fill Magic Dual Cement base and Fill Magic Dual Cement catalyst onto the spatulation block. Mix the base and the catalyst to a uniform color, i.e. approximately 10 seconds.

Place the mixture in the preparation, and if necessary, inside the facet. Secure the piece in the correct position and ensure absolute insulation. Light-cure for 3 seconds and then remove the proximal and cervical occlusive excess using a suitable tool.

After removing all excess, light-cure the Fill Magic Dual Cement on each of the dental faces for 40 to 60 seconds. Complete with the finishing step, checking the occlusion and promoting polishing using the ONESTEP Silicone Polishing Tips from Coltene.

2.6. Endodontic Core Cementation:

Perform absolute insulation. Promote the technique of acid conditioning on the enamel, according to clinical evaluation and the

manufacturer's technical recommendation. Apply one coat of One Coat Bond SL adhesive according to the instructions for use. Dispense equal parts of Fill Magic Dual Cement base and Fill Magic Dual Cement catalyst onto the spatulation block. Mix the base and the catalyst to a uniform color, i.e. approximately 10 seconds.

Apply the mixture on the surface of the core and inside the preparation. Properly position the core in the previously prepared location.

Light cure for 40 seconds in each direction and wait for 5 minutes for self-polymerization inside the preparation.

Note 3: Metal cores do not allow the passage of light, so part of the cement is only chemically activated.

3. Precautions, Restrictions and Warnings:

3.1. Precautions

- Fill Magic Dual Cement must not be used as lining or temporary restoration.
- It must not be used with products containing phenolic compounds (eugenol, clove oil, Copal varnish), since they inhibit Fill Magic Dual Cement polymerization.
- Avoid direct exposure of the material to sunlight or other sources of intense light.
- All dental material should only be used by dentists and authorized dental laboratories.
- Keep it out of the reach of children.

3.2. Restrictions and Warnings:

There are no contraindications when Fill Magic Dual Cement is used by a dental professional in accordance with the instructions for use. However, in patients who have a history of allergy to any of its components, we recommend not to use it.

Working time of 4 and 5 minutes is suitable for careful removal of excess material. Complete polymerization without light exposure requires more than 30 minutes. This is advantageous since any excess of material in the proximal areas becomes difficult to remove once the composite is completely polymerized.

4. Shelf life:

The expiration date on the packaging represents the maximum period for use of the product. Do not use after expiration date.

Under appropriate storage conditions, Fill Magic Dual Cement has a shelf life of 36 months.

5. Storage:

Store at room temperature 15 to 30° in dry

conditions and protected from light and heat exposure. Do not store together with products containing eugenol.

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