CIMENTO LS

PORTUGUÊS

Descrição:

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ENGLISH

Description:

Cement LS is indicated for cementing cast metal restorations, ceramic restorations, crowns, bridges and for use as a cement base or cavity liner.

Composition:

Powder: Zinc Oxide, Magnesium Oxide, Bismuth Carbonate, Silicon Dioxide and Titanium Dioxide

Líquido: Phosphoric Acid, Zinc Oxide, Aluminium Hydroxide and Water.

Indications:

- Fixing inlays, crowns and bridges.
- Application as a base or cavity liner.
- Provisional dental restorations.

Instructions For Use:

1. To mix the Cement LS use a cold glass plate (21°C) and stainless steel spatula.
 2. Remove dust from the bottle with a measuring spoon and remove excess of the spoon with a spatula.
 3. Put the powder on the glass plate and divide it into four equal portions.
 4. Divide one of the quarters in two equal parts.
 5. Dispense 3 drops of liquid Cement LS beside the powder and mix the lower portion with all the liquid in a circular motion for about 10 seconds.
 6. Add each portion divided above with liquid at 26 seconds intervals.
 7. After adding all portions of powder to liquid, mix more 30 seconds.
- Be sure that the powder was completely incorporated into the liquid in each successive step of the mixture.

The powder can be divided into smaller portions since the absorbed ranges of mixture and the total mixing time. The total mixing time is 1 minute and 30 seconds.

Time Control of Prey

Powder / Liquid Ratio:

A thick mixture becomes the prey of the material faster than a fine mixture. The faster the powder is added to the liquid, the faster will be hardening. An extended mix tends to increase the setting time. Using a standard proportion of powder and liquid, setting time will be the final 5 minutes and 30 seconds.

Restrictions:

There are no restrictions when the cement LS is used by a dental surgeon in accordance with the instructions.

However, in patients who have a history of allergy to components of this product, it is recommended not to use.

Validity and Storage:

Cimento LS is suitable for its specific application when stored at room temperature ambient (15°C - 30°C) and dry place and away from light and dust.

The expired date, which is in the package, show the maximum period the product should be used.

Presentation:

Cement LS Powder:

01 bottle with 28g
01 measuring spoon
01 instruction for use

Cement LS Líquido:

01 bottle with 10mL
01 measuring spoon
01 instruction for use

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