Magic BOND

PORTUGUÊS

Descrição

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ENGLISH

Description

Magic Bond is a light-curing bonding monocomponent adhesive, indicated for use in enamel.

Composition

BIS-GMA, TEGDMA, EDAB, initiator and stabilizer.

Indications

Magic Bond is indicated to improve the bonding of restorations in anterior and posterior teeth (acid conditioner technique) with light curing activation.

Techniques of Application

1. Install a rubber dam of another appropriate method to avoid contamination.

2. Clean the enamel surface using prophylactic paste (without fluoride and flavoring) and water.

3. Apply Magic Acid 37% Coltene (conditioner) on the enamel surface to be restored for 60 seconds in permanent teeth and 90 seconds in deciduous teeth.

4. Wash the conditioned dental surface with spray of water, free of essential oils that contaminate the surface.

5. Apply enough quantity of Magic Bond on the surface to be restored.

6. Spread Magic Bond with air streaming or fine brush over the whole surface.

7. Light-cure the Magic Bond for 20 seconds.

8. Apply the composite (ex: Fill Magic, NT Premium, Brilliant NG, etc).

9. Light-cure the composite for 20 or 30 seconds.

10. Finish and polish the procedure for restoration.

Contraindication

Magic Bond is used by surgeon dentist in accordance with the use instructions.

However in patients who possess description of allergy to the components of this product, do not use it.

Validity and Storage

Magic Bond must be stored at room temperature (15° - 30°C), free of dust, humidity and intense light. Do Not store with products which contain eugenol. The date of validity printed on the package, shows the maximum period to the use of this product.

Warnings

dental material must be used by authorizes dentists and dental laboratories.

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