MAGIC WHITE

PORTUGUÊS

Magic White

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ENGLISH

Magic White

Dentist-supervised take-home teeth bleaching system

1. Description

Magic White has been developed based

on highly recommended world-renowned products for bleaching of vital and non-vital teeth. Its formula presents the high viscosity of the Carbomer gel which does not leak from the tray, as other bleaching agents do. It uses Carbamide Peroxide as an active ingredient in order to obtain effective teeth whitening (bleaching). It has neutral pH so as not to cause the enamel and dentin decalcification. It also contains the powerful humectant (Glycerol) and a high-water content that helps prevent dental dehydration.

Magic White has crystalline appearance and it also has Potassium Nitrate in its formulation as a desensitizing agent, which reduces tooth sensitivity as much as possible during bleaching.

The length of treatment is at dentist's discretion.

Magic White bleaching gel is available in concentrations of 10%, 16% and 22%, to satisfy the professional's need.

2. Indication

Magic White is indicated for bleaching of vital and non-vital teeth based on the home-whitened dental whitening technique supervised by the dentist. Magic White effectively removes stains caused by congenital, systemic, pharmacological or traumatic factors, in addition to stains caused by aging and extrinsic pigment deposition.

3. Presentation

Magic White Replacement:

1 syringe of Magic White with 3g, available in 10%, 16% or 22% concentrations

1 applicator tip

1 instruction for use

Magic White Kit 1:

9 syringes of 3g each, available in 10%, 16% or 22% concentrations

9 applicator tips

1 instruction for use

Magic White Kit 2:

5 syringes of 3g each, available in 10%, 16% or 22% concentrations

5 applicator tips

2 1mm thick EVA plates for making trays

1 tray case

1 instruction for use

4. Composition

Carbamide Peroxide, Carbomer, Potassium Nitrate, Sodium Fluoride, Structure-Stabilizing

Agent (Glycerol), Neutralizing Agent and Deionized Water.

5. Instructions for Use

Before starting the procedure, wash and dry the tray that will be used in the treatment.

Clean the dental surface as instructed by the dentist.

After cleaning the dental surface, press the plunger of the syringe applying a small drop of Magic White gel to each tooth space in the impression tray (inner vestibular face of the impression tray of the dental elements to be treated).

Place the tray in the mouth so that it fits the teeth and the gel overlaps the dental surface.

Avoid and remove the excess of bleaching gel from dental surface using a soft toothbrush or your own clean finger.

Let the product act on the dental surface according to the time and frequency stipulated by the dentist (*).

After the treatment, take off the tray and rinse your mouth with plenty of water to completely remove the gel.

Wash and dry the tray before and after use.

Store the tray in a tray holder.

Use the product as instructed by your dentist.

Always warn your dentist about any discomfort during the treatment.

(* Treatment Suggestion Table:

Presentation	Time and Frequency
Magic White 10%	1 X 2 hours / day or night
Magic White 16%	1 X 90 minutes / day or night
Magic White 22%	1 X 45 minutes / day or night

Observation
Taking a break of one or two days, and/or reducing length of treatment is suggested for sensitive patients.

6. Precautions, Side Effects, Contraindications, Restrictions and Warnings

6.1. Precautions and Side Effects

The treatment with Magic White must be done only with prescription, indication and supervision of a dentist.

Avoid purposeful and excessive ingestion of the product. Ingesting large amounts of the product can cause nausea.

In case of ingestion, drink plenty of water

and seek medical care if you feel uncomfortable.

Avoid consuming acidic foods or drinks as they may cause increased sensitivity during whitening. Strongly colored drinks or foods should also be avoided so that the whitening process is not compromised.

In case of direct contact with the eyes, rinse with plenty of running water and seek medical care if you feel uncomfortable.

It is usually common to experience tooth sensitivity during the whitening treatment, mainly to variations in cold temperatures. In this case, if the sensitivity persists, tell your dentist so he will be able to prescribe tooth sensitivity treatments.

Gingival irritation can happen when using mouthwashes or even toothpastes with strong aggressive flavor before using the tray with the gel. It is recommended that, if you wish to use these products, wait for at least 30 minutes (or enough time for the disappearance of sensitivity) before using the gel. Aggressive gum brushing can also make it sensitive and result in some irritation.

Some patients may experience mild irritation of the gums, tongue or lips, usually due to excessive amount of gel in the tray.

In case of doubts and/or in the event of any problem, interrupt the treatment and notify your dentist immediately;

In case of allergic reactions, suspend the use of the product.

Whitening procedures are currently effective, efficient and safe, if carried out properly and with the correct materials.

Foods and drinks containing strong colors should be avoided for at least 48 hours after the treatment or consumed in moderation.

6.2. Restrictions and Warnings

Keep the product away from heat and/or sunlight.

Keep out of the reach of children;

Follow local legislation to dispose of the product.

Product not injectable.

Do not reuse the packaging.

Do not drink, eat or smoke during treatment.

Do not smoke right after treatment - wait for at least 2 hours.

6.3. Contraindications

The product must not be used by pregnant and/or lactating women, and patients under 15 years old;

Do not use Magic White if the patient is sensitive, allergic and/or intolerant to any of the components of the product.

Patient's oral health must be apt for the treatment to be performed. Do not use in case of dental caries, cracking, infiltrations in restorations, exposed dentin, gingival tissue inflammation, oral lesions and/or any other factors that may compromise the treatment, in order to avoid sensitivity, pain and/or more serious problems during the procedure.

7. Expiration

Magic White has an 18-month shelf life from manufacturing date, even after being opened, as long as it is kept in its original packaging and in adequate storage conditions.

The expiration date shown on the packaging represents the maximum period for using the product.

Do not use after the expiration date.

8. Storage Conditions

Keep the product closed and in its original packaging after use.

Store it at temperatures from 5°C to 28°C, in dry place, protected from direct sunlight and heat.

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