PORTUGUÊS

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ENGLISH

1. Product Description

TEMPORARY FILLING MATERIAL is a ready-to-use eugenol-free material for temporary fillings, that hardens quickly in the mouth when in contact with saliva. Optimized adhesion, ensuring good marginal insulation. It has no harmful effects on the gingiva and pulp (non-irritating product).

1.1 Use Indication

Indicated for temporary filling of dental cavities.

1.1 Composition

Zinc Oxide, Zinc Sulfate, Calcium Sulfate, Silicon Dioxide, Dibutyl Phthalate, Polyvinyl Acetate, Sodium Fluoride, Mint Flavor.

1.1 Contents

Pot with 25g of product.

2. Application Techniques

The required amount of TEMPORARY FILLING MATERIAL must be taken from the container and applied directly to the mouth, without mixing.

Hardening takes place when in contact with saliva. The product is a little dehydrating during the hardening process, which can cause slight sensitivity to dentin.

When cavities are too deep, it is not necessary to fill them completely. This facilitates further material removal before the next treatment. In exposed areas it is recommended to observe a 15-minute interval at most. The Temporary Filling Material curing time is between 15 and 30 minutes.

3. Cautions, Restrictions and Warnings

3.1 Precautions

Due to the product's sensitivity to moisture, keep using the stopper and silica after opening the bottle. After opening, keep the package hermetically closed, as the product absorbs moisture.

It must only be used by authorized dentists and laboratories.

Keep out of reach of children.

3.2 Restrictions and Warnings

There are no warnings if Temporary Filling Material is used by a dental professional in accordance with the instructions for use. However, we recommend not using it in patients who have a history of allergy to any of its components.

4. Expiration

The expiry date of Temporary Filling Material is 36 months. The expiry date shown on the packaging represents the maximum period for using the product. Do not use after the expiry date.

5. Storage Conditions and Transportation

TEMPORARY FILLING MATERIAL serves its intended purpose when stored at room temperature (15°C - 30°C), in dry place, protected from light and heat.

Under proper storage conditions TEMPORARY FILLING MATERIAL has a shelf life of 36 months.