PORTUGUÊS

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ENGLISH

1. PRODUCT DESCRIPTION

INNOVA FLOW is a highly flowable, light-curing nanohybrid composite resin with zirconia filler, to be applied to prepared dental cavities without the need for pressure or condensation, creating an intimate bond on the walls of the preparation. Once polymerized, its properties are similar to those of known nanohybrid composite resins. In small quantities, it can also be applied to characterize grooves, pigments or mask stains or dental discoloration that may influence the aesthetic result, as well as to opacify composite resin restorative surfaces or metal surface repairs.

2. INDICATION FOR USE

Aesthetic anterior and posterior restorations. Due to its lower viscosity, it is more easily applied to class III and V restorations. It is also suitable for minimally invasive cavity preparations (including small occlusal restorations and sealings, not subject to stress) and repairing marginal defects (infiltration) in previously existing restorations. It is also suitable for repairing small aesthetic defects in indirect restorations and resin or acrylic temporaries. Due to its fluidity and elasticity (lower filler content), it is ideal for the lining and base (initial layers) of class I and II restorations. Restoration of enamel damaged by erosion, abrasion or displacement; Application of an aesthetic layer on the enamel surface to hide defects and restorations. Sealing fossils and fissures. Filling sunken areas. Class V. Small class I, III and IV restorations. Direct resin veneers. Porcelain repairs. Cementation of porcelain veneers. Bonding of dental fragments. Restoration of structural defects: hypoplastic stains. Repair of marginal defects in composite resin restorations. Construction of small cores. Characterization and opacification of tooth structure.

3. PRESENTATION

Replacement: 01 Innova Flow plastic applicator syringe, in contents that can vary from 1.0g; 1.2g; 1.4g; 1.5g; 2.0g; 2.3g; 2.4g; 2.5g; 3.0g; or 4.0g, depending on the colors available + application cannulas + 01 Instruction for Use. • Kit: Of 02; 03; 04; 05; 06; 07; 08; 09 or 10 Innova Flow syringes, in contents that can vary from 1.0g; 1.2g; 1.4g; 1.5g; 2.0g; 2.3g; 2.4g; 2.5g; 3.0g; or 4.0g; depending on the colors available + application cannulas + 01 Instruction for use. • Blister: 01 blister, in contents that can vary from 1.0g; 1.2g; 1.4g; 1.5g; 2.0g; 2.5g; 3.0g; in the colors available for Innova Flow + 01 Instruction for Use. The colors available are: A1; A2; A3; A3.5; A4; B1; B2; B3; B4; C1; C2; C3; C4; D1; D2; D3; D4; W; TR; Bleach; OA3.5; A 0.5; B 0.5; OP; Transparent and Opalescent. Accessory: Cannulas for application (each Innova Flow syringe comes with 05 cannulas for applying the product). Note: Printed and/or digital, available via QR code printed on the packaging. In the case of digital printing, it can be requested physically through Vigodent Customer Service. Composition: Mixture of monomers (BIS-GMA, BIS-EMA, TEGMA, UDMA), BHT, photoinitiators, fillers, zirconia/silica and pigments.

4. HOW TO USE

Tooth surfaces should be cleaned before starting treatment; • Before isolating the tooth, select the appropriate shade of restorative material; • Promote absolute isolation; • Prepare the cavity using the usual technique; • Perform pulp protection, when necessary, with calcium hydroxide or glass ionomer lining material according to clinical assessment; • Perform the acid etching technique according to the manufacturer's instructions; • Use a Magic Bond DE adhesive according to its instructions for use; • Apply Innova Flow directly into the cavity with the applicator tip; • Do not apply layers thicker than 2mm in order to optimize polymerization; • Once the application is complete, pull back on the syringe plunger to prevent extra material from escaping; • Light cure for 40 seconds; • Use the conventional method and materials for sculpting; • Polish with silicone mounted tips or polishing and finishing wheels and sandpaper.

5. PRECAUTIONS, RESTRICTIONS AND WARNINGS

Precautions
This product is intended to be used only as described in the instructions for use. Any use other than that described in the instructions for use is the sole responsibility of the professional using it. Contact with saliva and blood during application of the composite may damage the restoration. The use of relative isolation with cotton protection or absolute isolation with rubber or another suitable isolation technique is recommended. As a preventative measure, to avoid a possible pulp reaction, use a calcium hydroxide cement, glass ionomer cement or other type of liner as a base before inserting the material into the cavity. Discoloration can occur when cationic mouthwashes are performed, as well as plaque and chlorhexidine developers. Restrictions and warnings
This resin is sensitive to light, so excessively intense ambient light can cause premature polymerization. Always keep the product closed when not in use. Once injected out of the packaging, it should be used quickly. Keep out of reach of children. It should not be used with products containing phenol (phenol, clove oil, copal varnish) or zinc oxide cements with eugenol, as they prevent or affect polymerization. After use, the material must be properly closed. All dental equipment should only be used by authorized dentists and dental laboratories. Syringes should be disposed of after use, together with the materials from the dental practice and/or laboratory. Contraindications
There are no contraindications when INNOVA FLOW is used by a dental professional in accordance with the instructions for use. However, patients with a history of sensitivity and/or allergy to any of the components are advised not to use it.

6. STORAGE

Must remain in its original, unopened packaging after use. Store at room temperature (15° to 30°C), in a dry place, protected from light and heat. Direct light on the product may alter the properties of INNOVA FLOW. If the product is stored under refrigeration, it must be removed from the refrigerator before use to return it to room temperature. Unsuitable storage conditions may compromise the product's performance.

7. SHELF LIFE

Under suitable storage conditions, INNOVA FLOW has a shelf life of 36 months, even after opening, provided it is kept in its original packaging and properly closed. The expiry date on the packaging represents the maximum period of use for the product. Do not use after the expiration date.

ESPAÑOL

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INNOVA FLOW is a high-flow, light-curing nanohybrid composite resin with zirconia filler, to be applied to prepared dental cavities without the need for pressure or condensation, creating an intimate bond on the walls of the preparation. Once polymerized, its properties are similar to those of known nanohybrid composite resins. In small quantities, it can also be applied to characterize grooves, pigments or mask stains or dental discoloration that may influence the aesthetic result, as well as to opacify composite resin restorative surfaces or metal surface repairs. repairs.

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