PORTUGUÊS

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ENGLISH

1. product identification

Ortho Kromm is a chromatic, charged orthodontic adhesive for fixing brackets. Ortho Kromm does not require mixing, is easy to apply and its composition promotes excellent adhesion of the bracket to the tooth, helping the professional at the moment of polymerization of the product, through the change of color.

2. Indications

Ortho Kromm is indicated for fixing metal brackets, especially for the straight wire technique and for fixing plastic brackets.

3. Composition

Mixture of monomers, photoinitiator, fillers and pigments.

4. Forms of presentation

Ortho Kromm: 01 Syringe containing 1.2g; 1.4g; 2.0g; 2.4g; 3.0g; 4.0g or 5.0g + 01 Instruction for Use

Ortho Kromm - Kit: 02; 03; 04; or 05;

Syringes, which may contain 1.2g; 1.4g; 2.0g; 2.4g; 3.0g; 4.0g or 5.0g in each syringe + 01 Instruction for Use

5. Instructions for use

- Clean the enamel surface using a prophylactic paste free of fluoride, flavorings or oil. Rinse with water and dry thoroughly with an air jet free of oil or moisture.
 - Relatively isolate with cotton rolls.
 - Apply Magic Acid 37% dental conditioner to the tooth surfaces, as indicated in the manufacturer's instructions for use.
 - Rinse with water and dry thoroughly with an air jet.
 - Apply a thin layer of Ortho Kromm to the base of the bracket and place it on the prepared area of the tooth.
 - First remove the excess before polymerization.
 - Once the bracket has been properly placed on the tooth surface, polymerize it on the cervical and incisal surfaces for 20 seconds each, using a halogen or LED light-curing device.
 - Remove any excess around the bracket using a fine finishing tip.
 - The wire can be leveled after curing.
- NOTE: Curing is indicated by the change in the color of the product.

6. Precautions, Contraindications and Warnings.

Precautions

FOR PROFESSIONAL USE

- Avoid direct exposure of the material to sunlight or other sources of intense light.
- Ortho Kromm should not be used or stored with products that contain phenol (phenol, clove oil, copal varnish) or zinc oxide and eugenol cements, as these materials prevent or affect polymerization.
- Discoloration may occur around the brackets when cationic mouthwashes, as well as plaque and chlorhexidine developers, are used over a long period.
- The Product must remain in its original, unopened packaging after use.

Warnings

- Keep out of reach of children.
- This product is intended to be used only as described in the Instructions for Use. Any use other than that described in the instructions for use is the sole and exclusive responsibility of the professional using it.
- All dental equipment should only be used by authorized dentists and dental laboratories.
- Syringes should be disposed of after use, together with the materials from the dental practice and/or laboratory.

7. Contraindications

There are no contraindications when Ortho Kromm is used by a dental professional in accordance with the instructions for use. However, patients with a history of allergy to the components of this product are advised not to use it.

8. Shelf life

Under proper storage conditions, Ortho Kromm has a shelf life of 36 months, even after opening, as long as it is kept in its original packaging and properly closed. The expiry date on the packaging represents the maximum period of use for the product. Do not use after the expiration date.

9. Storage

- Must remain in its original, unopened packaging after use.
- Store at room temperature (15° to 30°C), in a dry place, protected from light and heat.
- Direct light on the product may alter Ortho Kromm's properties.
- If the product is stored under refrigeration, it should be removed from the refrigerator before use to return it to room temperature.
- Unsuitable storage conditions may compromise the product's performance.

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