124 mm

Pasta de Polimento
Diamantada

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6.2. Warnings
Carry out at low pressure with low rotation, making intermittent movements to apply the product on the tooth. This prevents excessive surface heating.
6.3. Contraindications
There are no contraindications when Diamond Polishing Paste is used by a dental professional in accordance with the instructions for use. However, we recommend not using it in patients who have a history of allergy to any of its components, or when the adequate technique cannot be applied.
7. Expiration
• Diamond Polishing Paste has a 24-month shelf life, from the manufacturing date, even after opened, provided it is kept in its original packaging and under proper storage conditions.
• The expiration date displayed on the packaging represents the maximum period for using the product.
• Do not use it after the expiration date.
8. Storage Conditions
Keep the product in its original packaging always tightly closed. Store the product at temperatures from 5°C to 30°C / 41°F to 86°F. Do not freeze the product.

ENGLISH

1. Description
Diamond Polishing Paste is manufactured with extra fine granulation (2-4 microns) micronized diamond, for sole use by dentists or prosthetics. It yields excellent results in the polishing of porcelain, dental enamel, resins, composites and other restorative materials. Diamond Polishing Paste is a universal polishing paste.
2. Use Indications
Polishing Paste is indicated to be used with felt discs or velvet discs, which enhances its action in polishing and finishing restorative material in general.
3. Contents
Refill
01 syringe containing 2g of product
01 application tip
01 Instructions for use
4. Composition
Carbomer, Glycerin, Sodium Hydroxide, Diamond Powder, Deionized Water and Sodium Fluoride.
5. Instructions for use
The polishing paste is required to be used with felt or velvet discs, in a low rotation pen and contra-angle.
Carefully attach the disc to the contra-angle, making sure it is firmly attached. Dispense the necessary amount of product on the tooth so that it covers the area of the restoration. Then, polish the restoration making intermittent movements with the contra-angle on the dental element, in order not to cause local heating.
After the polishing is complete, simply rinse the tooth with water to remove the paste.
6. Precautions, Side Effects, Contraindications, Restrictions and Warnings
6.1. Precautions
Always use new felts for polishing. Using the same felt disc with abrasives from other pastes may compromise the polishing.

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6.2. Warnings

Carry out at low pressure with low rotation, making intermittent movements to apply the product on the tooth. This prevents excessive surface heating.

6.3. Contraindications

There are no contraindications when Diamond Polishing Paste is used by a dental professional in accordance with the instructions for use. However, we recommend not using it in patients who have a history of allergy to any of its components, or when the adequate technique cannot be applied.

7. Expiration

- Diamond Polishing Paste has a 24-month shelf life, from the manufacturing date, even after opened, provided it is kept in its original packaging and under proper storage conditions.
- The expiration date displayed on the packaging represents the maximum period for using the product.
- Do not use it after the expiration date.

8. Storage Conditions

Keep the product in its original packaging always tightly closed. Store the product at temperatures from 5°C to 30°C / 41°F to 86°F. Do not freeze the product.

ENGLISH

1. Description

Diamond Polishing Paste is manufactured with extra fine granulation (2-4 microns) micronized diamond, for sole use by dentists or prosthetists. It yields excellent results in the polishing of porcelain, dental enamel, resins, composites and other restorative materials. Diamond Polishing Paste is a universal polishing paste.

2. Use Indications

Polishing Paste is indicated to be used with felt discs or velvet discs, which enhances its action in polishing and finishing restorative material in general.

3. Contents

Refill

01 syringe containing 2g of product

01 application tip

01 Instructions for use

4. Composition

Carbomer, Glycerin, Sodium Hydroxide, Diamond Powder, Deionized Water and Sodium Fluoride.

5. Instructions for use

The polishing paste is required to be used with felt or velvet discs, in a low rotation pen and contra-angle.

Carefully attach the disc to the contra-angle, making sure it is firmly attached. Dispense the necessary amount of product on the tooth so that it covers the area of the restoration. Then, polish the restoration making intermittent movements with the contra-angle on the dental element, in order not to cause local heating.

After the polishing is complete, simply rinse the tooth with water to remove the paste.

6. Precautions, Side Effects, Contraindications, Restrictions and Warnings

6.1. Precautions

Always use new felts for polishing. Using the same felt disc with abrasives from other pastes may compromise the polishing.

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