# EDTA TRISSÓDICO GEL

## PORTUGUÊS

**Descrição**

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ENGLISH
---------

**Description**

EDTA GEL 24% is a chelating agent used inside the pulp cavity to prepare the walls of root canals. It acts to demineralize dentin particles, facilitating their dissolution and efficiently removing dentin sludge and scrapings, increasing dentin permeability and facilitating adherence of the treated connective tissue to the root surface in periodontal therapy.

**Presentation:**

EDTA GEL 24% - Replacement
01 Syringe of 1.4g, 2g, 3g or 4g;
03 applicator tips
01 Instructions for use
EDTA GEL 24% - Kit
02; 03 or 04 syringes, can be 1.4g; 2g; 3g or 4g; in each syringe
03 applicator tips (per syringe)
01 Instructions for use

**Composition:**

Disodium Ethylenediaminetetraacetic Acid (EDTA), Sodium Hydroxide, Deionized Water and Thickeners.

**Indication:**

- To facilitate endodontic instrumentation of root canals;
- Acts as a demineralizer of dentin particles by chelating calcium and magnesium ions.
- Aids in the cleaning of conduits prepared for the cementation of posts
- Coadjuvant in scraping the root surface.
- Root conditioning in periodontal surgery.

**Instructions for use:**

**In endodontics:**

- Connect the applicator tip to the syringe and draw a small amount of EDTA GEL 24% into the root canal;
- Leave the product to act for 2 to 3 minutes to begin instrumentration.
- The reaction with the dentin takes 5 to 6 minutes, after which time the product becomes less effective and neutralizes itself;
- Repeat the procedure, applying the same amount of product as necessary.
- Keep the canal moistened with EDTA GEL 24% throughout the procedure;
- At the end of the instrumentation, irrigate the canal using an irrigating solution such as sodium hypochlorite.

**In periodontics:**

- Remove granulation tissue;
- Apply EDTA GEL 24% and leave the product on for 2 to 3 minutes.
- Perform root scraping;
- Repeat the operation if necessary;
- Rinse thoroughly with saline solution;
- Suture if necessary;
- Rinse with saline solution or demineralized water;
- Place surgical cement, if necessary.

**On the prothesis:**

- Connect the applicator tip to the syringe.
- Introduce EDTA GEL 24% into the prepared canal and leave to act for 5 minutes;
- Irrigate with an irrigating solution such as sodium hypochlorite;
- Repeat the operation if necessary;
- Dry the canal with absorbent paper tips and

cement according to the technique indicated by the manufacturer of the chosen product.

**Precautions:**

- Completely remove 24% EDTA GEL before root canal restoration or temporary dressing
- Isolate the area before using the product.
- In case of direct contact with oral mucosa, rinse with plenty of water.
- In case of contact with eyes, rinse thoroughly with water. Seek medical help for evaluation and subsequent treatment.

**Warning:**

- FOR PROFESSIONAL USE
- Respect the expiration date. Never use the product if it has expired.
- Keep out of reach of children.
- Empty packaging must be disposed of in accordance with the rules for disposing of other materials, products and packaging from dental practices/laboratories.

**Contraindications**

- There are no contraindications when EDTA GEL 24% is used by a dental professional in accordance with the instructions for use. However, patients with a history of allergy to any of the components are advised not to use it, or when the stipulated technique cannot be applied.

**Shelf life:**

- Under appropriate storage conditions, EDTA GEL 24% has a shelf life of 36 months, even after opening, provided it is kept in its original packaging, properly closed and packaged in accordance with the storage instructions.

- The expiry date on the packaging represents the maximum period of use for the product. Do not use after the expiration date.

- Close the syringe immediately after use to prevent oxidation of EDTA GEL 24% .

**Storage:**

- Keep the syringe tightly closed immediately after use.
- Protect from moisture, heat and direct sunlight.
- Store the product at temperatures between 15°C and 30°C.

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