124 mm

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Gel de baixa viscosidade contendo ácido fluorídrico a 10%
Gel de baja viscosidad que contiene ácido fluorhídrico al 10%
Low-viscosity gel containing hydrofluoric acid at 10%

PORTUGUÊS

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ENGLISH

Hydrofluoric Acid is used in the acidic conditioning of porcelain and prosthetic parts aiming to facilitate and improve mechanical anchorage, thus obtaining better adhesion. When in contact with the prosthetic parts, it causes their etching, increasing surface roughness and consequently the surface area in order to improve adhesion.

Use Indications:
It is a 10%-concentrated hydrofluoric-acid-based gel, indicated for acid etching of porcelain in order to increase adhesion to restorations.

Contents:
Refill
01 syringe containing 2,0g of product
02 application tips
01 Instructions for use

Composition:
Distilled Water, Ultragel Carbomer, Hydrofluoric Acid and Pigment.

Instructions for use:
PORCELAIN REPAIR
1- Make a 2-3mm bevel, blast it with aluminum oxide and wash it with water spray.
2- Etch the porcelain with the Porcelain Conditioner for 4-5 minutes, wash it, rinse it well and dry it thoroughly.
3- Apply the *Silane Bonding Agent according to the instructions for use.
4- Apply the adhesive and composite according to the manufacturer's instructions.

PORCELAIN / METAL REPAIR
1- Prepare the porcelain (2-3mm bevel).
2- Blast the metal and porcelain with aluminum oxide, wash it with water spray and dry it.
3- Apply Hydrofluoric Acid Conditioner uniquely to the porcelain surface for 4-5 minutes, wash it, rinse it well and dry it.
4- Apply the adhesive and composite according to the manufacturer's instructions.

HYBRID COMPOSITE REPAIR
1- Make a 2-3mm bevel, blast it with aluminum oxide and wash it with water.
2- Apply Porcelain Conditioner onto the restoration for 60 seconds, wash it with water spray and dry it completely.
3- Apply *Silane Bonding Agent as directed.
4- Apply the adhesive and composite according to the manufacturer's instructions.

PREPARATION OF PORCELAIN VENEERS
1- Isolate the outer face of the porcelain veneer with wax.
2- Apply Porcelain Conditioner to the inner face of porcelain veneer for 4-5 minutes, wash it thoroughly with water spray or ultrasonic bath and dry it.
3- Apply *Silane Bonding Agent as directed.
4- Cement with a *fluorine resin cement, according to the instructions for adhesive cementation of porcelain veneers.
Products sold separately.

Precautions, Side Effects, Contraindications, Restrictions and Warnings
For dental use only.
For professional use.
Keep out of the reach of children.
Prolonged use may cause overconditioning of the prosthetic part, that is, excessive conditioning of the prosthetic part, damaging it.
The use of PPE (Personal Protective Equipment) is required during its manipulation. The Dental Surgeon or Assistant must wear: goggles, cap, mask, gloves and a lab coat.
Allergic reactions or burns may occur in case of inadvertent patient contact.
Application tips (commonly found for purchase). After using the product, the application tip must be removed and discarded. The use of application tips other than those recommended by the manufacturer is of sole responsibility of the professional.

Restrictions and Warnings:
High temperatures may compromise performance and change product consistency. This product contains hydrofluoric acid. Avoid contact with enamel, dentin, eyes and skin. -- If contact occurs, wash with sodium bicarbonate saturated water. If eye and skin irritation persist, seek medical attention.
There are no special handling conditions. Dispose of the product according to local legislation.

Contraindications:
There are no contraindications when Hydrofluoric Acid is used by a dental professional in accordance with the instructions for use. However, we recommend not using it in patients who have a history of allergy to any of its components, or when the adequate technique cannot be applied.

Expiration:
Hydrofluoric Acid has a 24-month shelf life, from the manufacturing date, even after opened, provided it is kept in its original packaging and under proper storage conditions. The expiration date displayed on the packaging represents the maximum period for using the product.
Do not use it after the expiration date.

Storage Conditions:
Keep the product in its original packaging closed after use.
Store the product at temperatures from 5°C to 28°C, in dry place, protected from direct sunlight and heat.

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