PERFIL LAB

PORTUGUÊS

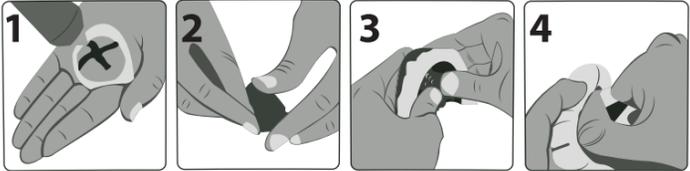
Imagem 1 e 2:

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ENGLISH

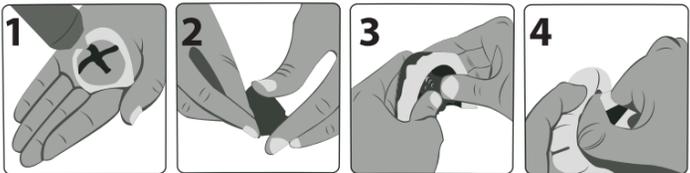
Image 1 and 2:

Using the measuring spoon, scoop the desired amount of PERFIL LAB from the jar, ensuring the spoon is properly filled with no empty spaces or excess material. Press the PERFIL LAB against the palm of your hand and mark the diameter of the measuring spoon on the PERFIL LAB.

For each spoonful of PERFIL LAB used, distribute the PERFIL CATALYST in an amount equivalent to two marked diameters in an "X" shape.

Image 3 and 4:

Mix PERFIL LAB and PERFIL CATALYST with your fingertips for 30 seconds, until a uniform color is achieved.

PERFIL LAB®

Composition: Polydimethylsiloxane, Corn Starch, Vaseline, Talc, Silicon Dioxide.

Indications: Control wall for metal structure planning, prosthesis resin filling, fabrication of provisionals, removable gingival mask reproductions, mock-up creation, isolation of teeth/plaster in the flask.

Characteristics: Soft, great heat resistance, high dimensional stability, excellent copy fidelity, extra hardness obtained after full curing. 85° SHORE A - after 24 hours at 23°C.

Warnings/Precautions: Avoid contact with eyes. We recommend using protective goggles for professionals during use. PERFIL CATALYST is an irritant. Keep the product out of reach of children. Single-use product. There are no restrictions when the product is used by professionals or authorized laboratories according to the Instructions for Use.

For professionals with a history of allergies to one or more product components, it is recommended not to use the product.

Exclusive use for dental laboratories.

Shelf life: 3 years from the date of manufacture.

For best results, use with PERFIL CATALYST.

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